

PRECAUTIONS:

For external use only • Do not use for purposes other than intended • Do not share your **FUSION® Allergy Cooling Mask** • Do not apply to sensitive skin or open wounds • Do not ingest contents • Seek medical advice if symptoms persist or worsen • Inspect your **FUSION® Allergy Cooling Mask** and discard it if damaged • Keep out of reach of children • The head straps in this product contain latex • Store at room temperature • Adult supervision is always required when applying the mask to children.

After treatment, it may be normal to experience some blurring in your vision. The blurriness should clear within a few minutes. If it does not clear within an hour, consult your eye care specialist.

FUSION® Allergy RANGE OF PRODUCTS



The **Fusion® Allergy** range (each sold separately). For best advice on which products are suitable for you, consult a medical professional or speak directly to Scope.



IRELAND
SCOPE

Suite 5, Westland House, Westland Park, Willow Road,
Dublin 12, Ireland


T: 1800 816 005 E: hello@fusionallergy.ie
www.fusionallergy.ie

UNITED KINGDOM
SCOPE

Unit 4, Amberley Court, County Oak Way, Crawley,
RH11 7XL, United Kingdom

T: 0808 175 3200 E: hello@fusionallergy.co.uk
www.fusionallergy.co.uk

FUSION® is a trademark of Scope Ophthalmics

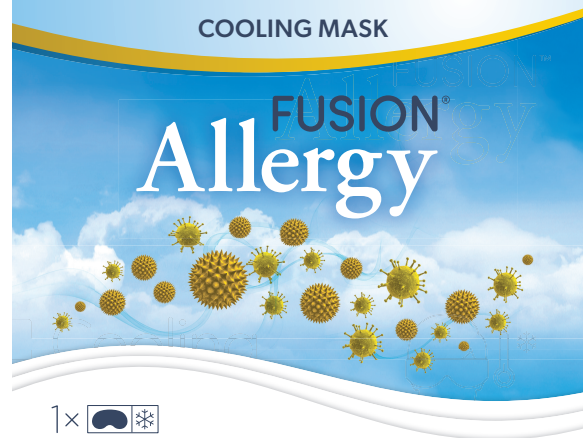

Hayfever Symptom Relief

Reusable cooling mask to relieve hayfever or allergic symptoms around the eyes, such as itchy, red, watery eyes as well as puffy or inflamed eyelids.

Featuring **HydroCool Technology**



Pollen



Dust Mites



Grass



Animal Dander




Suitable for children and adults

SCOPE

FUSION® Allergy Cooling Mask

The **FUSION® Allergy Cooling Mask** is a reusable cooling mask designed to relieve hayfever or allergic symptoms around the eyes, such as itchy, red, watery eyes as well as puffy or inflamed eyelids.

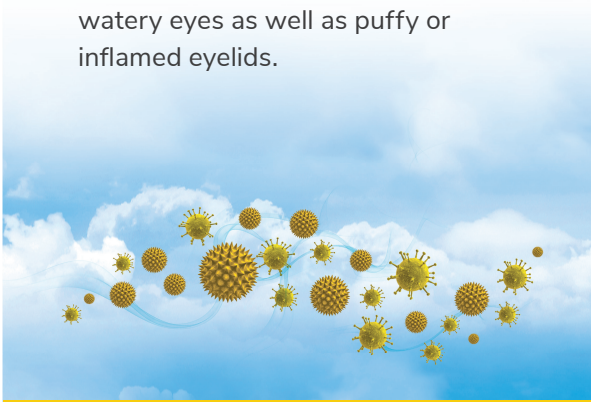

DIRECTIONS FOR USE:

1



Place the **FUSION® Allergy Mask** in a sealed plastic bag and place in the freezer for one hour.

2



Position the **FUSION® Allergy Mask** around the head and close eyes for 10 minutes adjusting the strap for comfort.

Do not wear the **FUSION® Allergy Cooling Mask** for longer than 20 minutes.

The **FUSION® Allergy Cooling Mask** features HydroCool Technology to provide cooling therapy to the closed eyelids.

RECOMMENDED USE:

Cold therapies can reduce the symptoms of hayfever or allergic symptoms around the eyes, such as itchy, red, watery eyes as well as puffy or inflamed eyelids.

USEFUL INFORMATION:

Contact lens wearers must remove lenses prior to using the **Fusion® Allergy Cooling Mask**.

Wipe immediately if exposed to foods, oils, grease or other dirt. Air dry for a minimum of 24 hours until it is completely dry before next use.

Latest revision date: October 2023