

Instructions for Use – please read carefully

FUSION® Allergy Eye Drops

Read this information carefully before you start using **FUSION® Allergy Eye Drops**. **DO NOT THROW THIS LEAFLET AWAY!** It contains important information. If you have any questions about **FUSION® Allergy Eye Drops** please ask your healthcare provider, pharmacist or Scope representative.

1. What is FUSION® Allergy Eye Drops and what is it used for

FUSION® Allergy Eye Drops is a Medical Device which contains Ectoin®, a natural cell-protective molecule with membrane-stabilizing properties.

FUSION® Allergy Eye Drops is an isotonic solution and can be used to alleviate and treat symptoms of allergic conjunctivitis. Symptoms like itchy, red, or watery eyes can effectively be reduced.

FUSION® Allergy Eye Drops protects against the harmful influences of allergens and supports the regeneration process of irritated and sensitive conjunctiva.

FUSION® Allergy Eye Drops is suitable for wearers of contact lenses and for sensitive eyes.

FUSION® Allergy Eye Drops is preservative free.

2. Warnings and precautions

- **FUSION® Allergy Eye Drops** should not be used in the case of hypersensitivity to Ectoin® or any of the other ingredients of the eye drops.
- If allergic reactions due to the product occur, the use of **FUSION® Allergy Eye Drops** must be discontinued immediately.
- After usage of one ampoule, the remaining liquid must be discarded, it is not intended for later usage. Do not use damaged or punctured ampoule.
- **FUSION® Allergy Eye Drops** is filled in small plastic parts that could be swallowed by small children. Danger of suffocation. Therefore, **FUSION® Allergy Eye Drops** should be kept out of reach of children.

3. What you need to know before using FUSION® Allergy Eye Drops

There are no data available regarding interactions between **FUSION® Allergy Eye Drops** and other eye drops or topical eye ointments, therefore the usage with other ophthalmological treatments is not recommended.

4. How to use FUSION® Allergy Eye Drops

Always use **FUSION® Allergy Eye Drops** according to these instructions for use. Consult your healthcare provider or pharmacist before use if you are not sure.

If not otherwise prescribed by your healthcare provider, please apply 1-2 drops per eye, several times a day. If necessary, **FUSION® Allergy Eye Drops** can be used multiple times per hour.

1. Detach one ampoule from the strip and open it by twisting off the tab.
2. Bend your head backwards, pull the lower eyelid downwards with your forefinger and look up.
3. Gently squeeze the ampoule together using the other hand and let one to two drops fall into the pocket of the eye.
4. The unit should not touch the eye. Close your eye slowly to disperse the solution throughout the eye where possible.
5. Repeat the procedure on the other eye.

5. Possible side effects of FUSION® Allergy Eye Drops

FUSION® Allergy Eye Drops is well tolerated. Up to now, no regularly occurring or persisting side effects are known.

In isolated cases, FUSION® Allergy Eye Drops may provoke short term eye irritation, ocular hyperaemia, or conjunctivitis.

Please inform your healthcare provider, pharmacist, or Scope representative if you notice side effects that are associated with the use of FUSION® Allergy Eye Drops.

If conditions persist or worsen, contact your healthcare provider or pharmacist.

6. How to store FUSION® Allergy Eye Drops

The expiry date of FUSION® Allergy Eye Drops is printed on the box and on the ampoule.

FUSION® Allergy Eye Drops must not be used after the expiry date has passed.

FUSION® Allergy Eye Drops should be stored between 2 °C – 25 °C.

7. Additional information

If recommended by your doctor/health care professional, FUSION® Allergy Eye Drops may be used after eye/ocular surgery and trauma.

If FUSION® Allergy Eye Drops is applied within its intended use, there is no evidence for a particular risk during pregnancy or breast-feeding.

Children 10 years of age and younger and patients unable to use FUSION® Allergy Eye Drops on their own should be assisted during the application of FUSION® Allergy Eye Drops.

FUSION® Allergy Eye Drops is suitable for use in children.

8. Content of FUSION® Allergy Eye Drops

15 x 0.5ml ampoules

Ingredients: Ectoin®, Sodium Chloride, Water, and Citrate Buffer

9. Legal Manufacturer



bitop AG
Stockumer Str. 28,
58453 Witten, Germany



SCOPE

UNITED KINGDOM

SCOPE

Unit 4, Amberley Court,
County Oak Way, Crawley,
RH11 7XL, United Kingdom

T: 0808 175 3200

E: hello@fusionallergy.co.uk
www.fusionallergy.co.uk

IRELAND

SCOPE

Suite 5, Westland House,
Westland Park, Willow Road,
Dublin 12, Ireland

T: 1800 816 005

E: hello@fusionallergy.ie
www.fusionallergy.ie

10. Date of information:

October 2023

EXP or



Use by date
(Expiry date)



Please do not use damaged
or punctured ampoules



Medical Device