






INSTRUCTIONS FOR USE


OPTASE® Comfort Drop

Applications:

OPTASE® Comfort Drop in the form of eye drops can be used to alleviate symptoms of irritation, dryness and fatigue in the eye. The special formula contains sodium hyaluronate, a naturally occurring substance in the eye, which forms an even, protective layer of film which can absorb large amounts of water and is resistant to evaporation. This helps hydrate the eye, improving visual comfort and making contact lenses easier to wear. The solution is highly recommended for contact lens users who frequently experience dryness or irritation in the eye.

Directions for use:

	Wash hands with water and soap prior to using the product.
	Open the bottle of moisturising solution by twisting the cap.
	For direct use: Place one or two drops in each eye and blink several times.
	For use with contact lenses: Apply one or two drops to the internal face of the lens prior to placing it on the eye.
	OPTASE® Comfort Drop may be used as often as is needed.

 UNOLAB MANUFACTURING, S.L.
Av/ De las flores 6, 28970
Humanes de Madrid (Madrid), Spain



0318

STERILE A

300448

Warning:

- Keep out of reach of children.
- This product does not contain preservatives. Use immediately once opened. Discard any remaining solution after use, as this may become contaminated, which may cause infection.
- Once opened, do not share a vial with anyone else.
- Do not use product if past its expiration date.
- Do not use if the product appears damaged.
- Do not use the product if you have any allergy to any of its ingredients.
- Should irritation occur, consult your eye specialist.
- Do not store at temperatures above 25°C.

Composition:

Sodium Chloride, Boric Acid buffer/Sodium Borate with 0.1% Edetate Disodium and 0.1% Hyaluronic Acid.

Distributed by

SCOPE

Scope Ophthalmics

Unit 4, Amberley Court,
County Oak Way, Crawley,
RH11 7XL, United Kingdom

Telephone UK: 0800 270 0253

Telephone Ire: 1800 816 005

Email: info@scopeeyecare.com

www.scopeeyecare.com

REV: 2010/DIC/03