

A Guide to Administering Eye Drops

Using the HYLO[®] COMOD bottle

Instructions for use

1




Always **wash your hands** before use.

5



Support the hand holding the bottle with your other hand for better control.

2

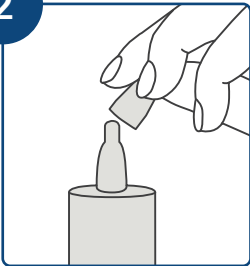

Remove the protective cap from the top of the bottle. **Do not throw away** the cap as you will need to replace it after use.

6




With your head leaning back slightly, gently pull on your lower eyelid. Press the base of the bottle to release the drop. Take care not to touch the tip of the bottle against your eye or eyelids.

3




The bottle must be prepared **before first use only**. Turn the bottle upside down and pump the base 6-10 times until the first drop appears. The bottle is now ready for use.

7

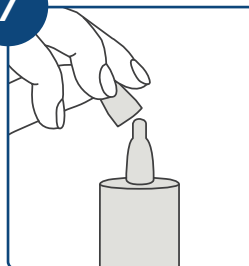

Replace the cap on top of the bottle after use.

4

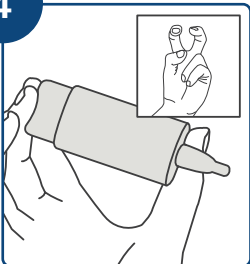

Hold the bottle with **the COMOD grip**. Put 2 fingers on the base of the bottle and your thumb on the shoulder.

Apply one drop to each eye up to three times per day. If you need to apply the drops more frequently, please speak to a Healthcare Professional.